

INSTALLATION INSTRUCTIONS

E.L. Mustee & Sons, Inc. 5431 West 164th St. Brook Park, OH 44142


Phone: 216.267.3100 Fax: 216.267.9997 Email: info@mustee.com Web: www.mustee.com



CareGiver Straight & Angled Grab Bars



READ ALL INSTRUCTIONS CAREFULLY AND INSPECT PRODUCT FOR DAMAGE BEFORE STARTING YOUR INSTALLATION

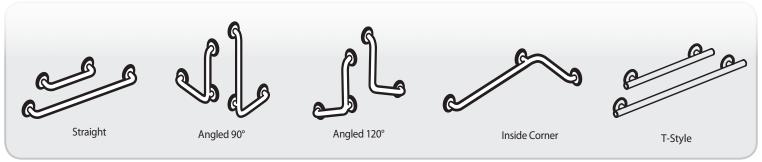

INCLUDED IN CARTON:

One (1) straight CareGiver® grab bar One (1) set of installation instructions

TOOLS AND MATERIALS REQUIRED FOR INSTALLATION:

Electric drill

1/8" wood or metal drill bit

1/4" masonry drill bit (Only needed if installing on a tiled wall. See instructions, Step 1)

Tape measure

Screwdriver

Level

Eight (8) #12 x 2" pan head self-tapping screws

PREPARATION:

Check local code for proper placement of Grab Bar to meet requirement such as ADA or ANSI 117.1 Consumer Safety Requirements.


Before installing the Grab Bar you wil have to prepare the area.

Rough-in Installation: Determine where the Grab Bar will be mounted and install bracing to the rough-in framing to support the Grab Bar through the shower wall.

Existing Wall Installation: Locate stud behind the existing shower wall to determine where the Grab Bar can be safely mounted.

Clean that portion of the shower wall where the Grab Bar will be installed.

Please read the instructions before proceeding with the installation.

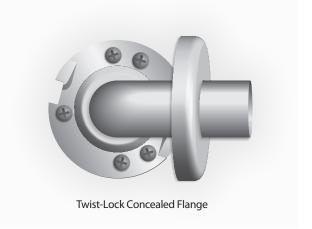



CareGiver*Grab Bars are only as strong as the strength and rigidity of the material to which they are mounted. They must be mounted through shower wall and screwed directly to wood or metal stud.

INSTALLATION

STEP 1:

Position mounting flanges and mark where holes need to be placed. Drill 1/8" pilot holes through the shower wall into wood or metal stud.


NOTE: If wall is tiled, drill through tile (only) with a 1/4" dia. masonry or ceramic tile drill and then change to the 1/8" drill to complete.

STEP 2:

Install mounting screws (#12 x 2" Pan Head – not supplied) through grab bar mounting bracket and directly into the wood or metal stud or bracing.

STEP 3:

Once the grab bar is securely fastened to wall, align the cover plate locking tabs with the slots in the flange. Turn the "Twist-Lock" cover plate clockwise approx. 1/4" turn to lock in place.

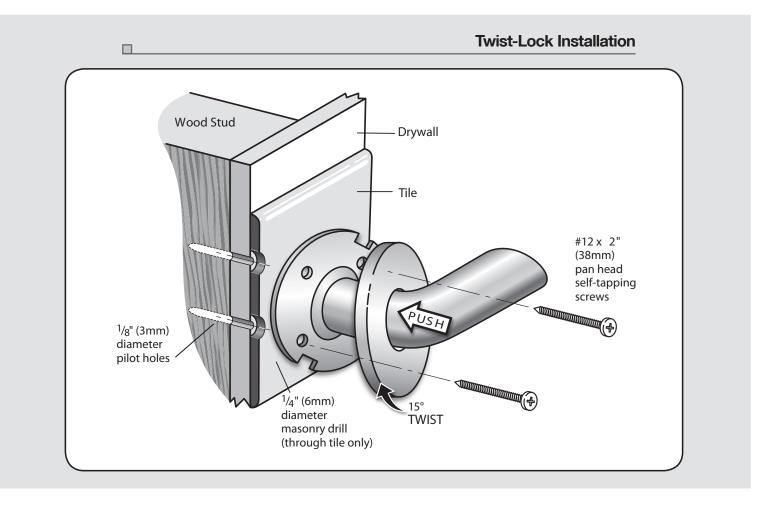

CARE & CLEANING

To clean, wipe bars with any non-corrosive household cleaner.

INSTALLATION QUESTIONS

Questions on the installation or missing parts, call 216-267-3100 or e-mail <u>customerservice@mustee.com</u> for assistance. Hours of operation are Monday to Friday, 8:30 a.m. to 4:30 p.m. E.S.T.

Please have the installation manual available when calling. DO NOT RETURN PRODUCT TO THE PLACE OF PURCHASE.

